

The Tempus Nasal Sample Collection Kit

WARNINGS

Individuals exhibiting severe COVID-19 warning symptoms are not allowed to proceed with kit registration and instead directed to seek emergency medical care.

This product has not been FDA cleared or approved but has been authorized by FDA under an Emergency Use Authorization for use by Tempus Laboratories located at 600 W Chicago, Ste 510, Chicago, IL 60654, which is certified under CLIA and meets the requirements to perform high complexity tests.

This product is authorized only for the detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens.

The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

Federal law restricts this device to be sold by or on the order of a licensed practitioner.

This Home Nasal Swab Collection Kit is intended for 18 years old or older individuals to self-collect anterior nares specimen unsupervised at home when determined to be appropriate by their Healthcare provider.

Intended Use

The iC SARS-CoV-2 Test is a reverse transcription, real-time polymerase chain reaction (RT-PCR) test intended for the qualitative detection of nucleic acid from SARS-CoV-2 nasopharyngeal (NP), anterior nares (AN or anterior nasal), mid-turbinate nasal, and oropharyngeal (OP) swab specimens collected from individuals suspected of COVID-19 by their healthcare provider.

This test is also for use with AN swab specimens that are self-collected unsupervised at home by individuals 18 years of age or older using the Tempus Nasal Sample Collection Kit, when determined to be appropriate by their healthcare provider. Emergency use of this test is limited to the authorized laboratory.

Interfering Substances

The iC SARS-CoV-2 Test uses the primers/probes included in the Thermo Fisher TaqPath COVID-19 combo kit, and ThermoFisher has granted a right of reference to the interfering substances information. There are no known interfering substances. Please refer to the TaqPath COVID-19 combo kit instructions for use for additional information.

'TEMPUS

600 West Chicago Ave, Ste 510
Chicago, IL 60654

[Tempus.com](https://tempus.com) | 800.739.4137

Tempus Labs, Inc. | All Content Copyright 2021 | 03.24.21

'TEMPUS

NASAL SAMPLE COLLECTION KIT

R_x Only

IVD

For Use Under Emergency Use Authorization (EUA) Only

For individuals 18 years or older


PRE-COLLECTION

1 | Visit the FedEx website at [fedex.com/labreturns](https://www.fedex.com/labreturns) to view drop box locations and pick up schedules.

- Your sample should be shipped the same day that you collect it. Make sure you are able to get to your drop box before the last pickup time on the same day you collect your sample.
- Do not deliver your sample to a drop box on the weekend.


2 | Open test kit on a clean surface.

3 | Identify your sample transport tube.

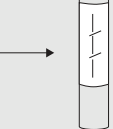- Using a permanent pen, write your date of birth (MM-DD-YY) on the enclosed label. 
- Remove the adhesive backing from the label sticker and affix it to the sample transport tube.


Note: Do not write on or cover up the printed barcode or 9 digit ID number on the tube.

4 | Activate your Tempus kit at activate.tempus.com.

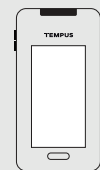

Follow the instructions on the fill out the form. Failure to write the appropriate information may result in your sample not being tested.


5 | Wash your hands thoroughly for 20 seconds with soap and water before collecting your sample.

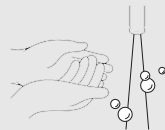

COLLECTING YOUR SAMPLE


6 | Prepare the sample transport tube by tapping the tube gently and ensure the liquid is at the bottom.

- Unscrew the cap from the sample transport tube and place it on the table.

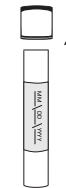


7 | Remove the swab from the package.

- Do not touch the soft tip of the swab or lay the swab down on any surface.

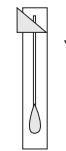

8 | Insert the soft tip of the swab into one nostril, until the tip of the swab is no longer visible.

- Rotate the swab four times in a circular motion around the entire inside edge of the nostril then keep it in place for 15 seconds.
- Repeat the same process in the other nostril using the same swab.

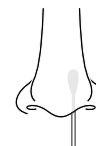

Note: Simply twirling the swab against one part of the inside of the nose, or leaving the swab in the nose for 10–15 seconds, is not proper technique and may result in an insufficient sample.

9 | Place the swab (tip first) into the sample transport tube.

- Tip of swab will be in the liquid. Leave the swab inside the tube.
- If the swab is too long to fit inside the tube, break the swab handle against the side of the tube at its perforated break point. Discard the top portion of the swab.




10 | Screw the cap onto the sample transport tube until it is closed tightly.



RETURNING YOUR SAMPLE

11 | Confirm that the sample transport tube is tightly closed and has your date of birth on the transport tube.

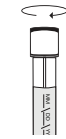

12 | Place the sample transport tube in the biohazard specimen bag and seal the bag tightly.



Make sure to leave the absorbent sheet inside the biohazard specimen bag.


13 | Wipe the outside of the bag with the provided alcohol pad. Dispose of wipe.

14 | Place the biohazard specimen bag into the Tempus-labeled box it came in. Place the Tempus-labeled box into the FedEx UN 3373 Pak.



Remove the adhesive liner of the FedEx UN 3373 Pak and press down firmly to seal the Pak tightly.

15 | Drop off UN 3373 Pak with shipping label at the nearest FedEx Drop Box (see item 1 for locations).

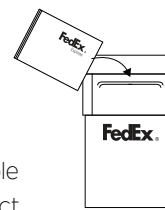

- Please drop off your sample on the same day you collect it and before the last pickup time.
- Do not deliver your sample to a drop box on the weekend.

✔ YOU'RE ALL SET.

After your sample has been dropped off at FedEx, your order will be processed once received.