

# The SalivaDirect funnel collection kit.

## WARNING AND PRECAUTIONS BEFORE BEGINNING

Before providing your sample, first carefully read through the instructions below about how to provide a good saliva sample. **Not following the instructions could lead to an incorrect result.**

It is very important that you do not force yourself to produce a saliva sample. Please do not cough anything up from your chest. Please do not sniff anything back from your nose into your mouth.

Do not eat, drink, smoke, clean your teeth, chew gum, or use nasal sprays for 30 minutes before collecting your saliva sample.

If you have any questions, please reach out to the support email on the activation card.

This product has not been FDA cleared or approved, but has been authorized for emergency use by FDA under an EUA; This product has been authorized only for the collection and maintenance of saliva specimens as an aid in the detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens; The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of medical devices under Section 564(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.



**Rx Only**  
IVD  
For Use Under Emergency Use Authorization (EUA) Only  
For individuals 18 years or older

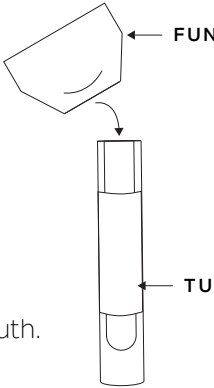
## PRE-COLLECTION

This test is designed to test saliva that naturally collects in your mouth (for example when you think about a delicious meal).

**WARNING: Do not eat, drink, smoke, clean your teeth, chew gum, or use nasal sprays for 30 minutes before collecting your saliva sample.**

- ① Visit the FedEx website at [fedex.com/labreturns](https://www.fedex.com/labreturns) to view drop box locations and pick up schedules.
  - Your sample should be shipped the same day that you collect it. Make sure you can get to your drop box before the last pickup time on the same day you collect your sample.
  - Do not deliver your sample to a drop box on the weekend.
- ② Identify your sample transport tube.
  - Using a permanent pen, write your date of birth (MM-DD-YY) on the patient identifier sticker.
  - Remove the adhesive backing from the patient identifier sticker and affix it to the sample transport tube.
  - Do not write on or cover up the printed barcode or ID number on the tube.
- ③ Activate your kit using the link on the registration card.
  - Follow the instructions on the page to fill in your information.


## COLLECTING YOUR SAMPLE

- ④ Wash your hands with soap and water for at least 20 seconds or clean them with alcohol-based sanitizer.
- ⑤ Remove everything from the collection kit and place it in front of you. Set the bag aside. **Avoid touching the inside of the funnel.**
- ⑥ Pick up the funnel and the sample tube. Position the funnel into the collection tube as shown.

The diagram shows a funnel being inserted into the top of a sample tube. An arrow points to the funnel with the label 'FUNNEL', and another arrow points to the tube with the label 'TUBE'.
- ⑦ Swallow once to clear your mouth.
- ⑧ Imagine eating your favorite food, think about an upcoming meal or even a sour lemon and allow saliva to collect in your mouth. Be patient and it will start to collect!
  - Remember to avoid coughing or sniffing.
- ⑨ Hold the funnel up so that it surrounds your mouth. Then gently release the saliva that has pooled in your mouth into the funnel.
  - Please avoid 'spitting' into the tube. Saliva should instead be gently released from your mouth.
- ⑩ Collect enough saliva so the tube is half-filled. Any bubbles in the tube do not count towards the final volume.
  - You may have to repeat this process a couple of times until the tube is half-full.
  - Be patient and don't force it. Just keep thinking about food and you'll get there!

- ⑪ Once you have collected enough saliva, remove the funnel from the sample tube and tightly screw the cap back onto the tube.
  - Make sure the cap is secure before proceeding to the next step.
- ⑫ Use the alcohol wipe to wipe your hands and the sample tube. DO NOT wipe the printed part of the label.
- ⑬ Place the closed sample tube into the biospecimen bag.
- ⑭ Discard the alcohol wipe and used funnel.
- ⑮ Wash your hands thoroughly with soap and water or clean them with alcohol-based sanitizer.

## RETURNING YOUR SAMPLE

- ⑯ Place the biospecimen bag into the SalivaDirect-labeled box it came in. Place the SalivaDirect-labeled box into the FedEx UN 3373 Pak.

The diagram shows a rectangular box with the FedEx Express logo on the top surface. The box is shown from a three-quarter perspective, with the top flaps slightly open.

  - Remove the adhesive liner of the FedEx UN 3373 Pak and press down firmly to seal Pak tightly.
  - Please drop off your sample on the same day you collect it and before the last pickup time. Do not deliver your sample to a drop box on the weekend.