


Clinic guide to ordering Tempus tests

Your comprehensive guide to understanding our molecular test offerings, interpreting our test names, and streamlining the ordering process.

TEST NAME	TURNAROUND TIME	CLINIC SPECIMEN NEEDED	SPECIMEN COLLECTION KIT(S)
xT CDx DNA seq <i>T is for Tissue</i>	~9 days after specimen receipt	Blood or Saliva Tempus requests a tissue sample from Pathology separately	
xR RNA seq <i>R is for RNA</i>	~9 days after specimen receipt	None from clinic Tempus requests a tissue sample from Pathology separately	N/A
xF/xF+ liquid biopsy <i>F is for Fluid</i>	~7-9 days after specimen receipt	Blood	
xT and/or xR for hematologic malignancies	~10-21 days after specimen receipt, and when xT and xR are ordered in combination	Blood or Bone Marrow Aspirate The Tempus kit for hematologic malignancies is orange.	
CancerNext® or CancerNext-Expanded®, +RNAinsight®, BRCPlus® hereditary cancer testing Powered by Ambry Genetics®, a Tempus company	CancerNext® or CancerNext-Expanded®: ~5-14 days after specimen receipt BRCPlus®: ~4-7 days after specimen receipt	Blood or Saliva Draw the EDTA blood tube first, then if ordering +RNAinsight, draw the PAXgene® blood tube last	 <p>Tempus offers three hereditary kits. Each gold kit displays the specimen type: DNA (blood), DNA (saliva), or DNA + RNA (blood).</p>
xM MRD for CRC <i>M is for Monitoring</i>		Blood Royal Blue-colored xM kit (Please note: Check the test name on each kit carefully, as blood kits look similar)	
xM NeXT Personal® Dx for solid tumors <i>M is for Monitoring</i>		Blood Tempus requests a tissue sample from Pathology separately	

FINANCIAL ASSISTANCE

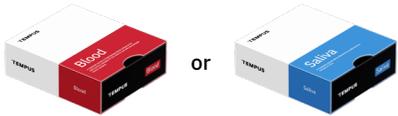

tempus.co/patient-forms

ORDERING A TEST



tempus.co/test-ordering

ORDERING SUPPORT



tempus.co/care-team-guide

MOBILE PHLEBOTOMY



tempus.co/phlebotomy

TEMPUS | ONCOLOGY

This is AI-enabled precision medicine.
This is the future of healthcare.

tempus.com/oncology/ 